


Anytime
Continuous Glucose Monitoring System

CT15/CT3 Series

Quick Start Guide


Parts

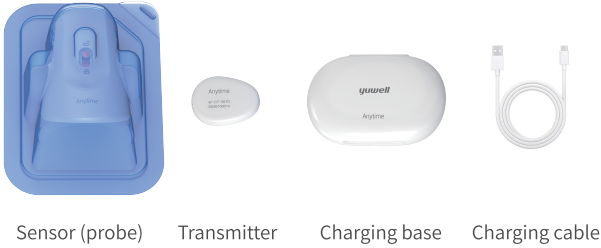

1 Preparations

Step 1 Download the software

- Turn on the mobile Bluetooth
- Scan the QR code below through WeChat to download or update the App
- Follow the prompt to log in

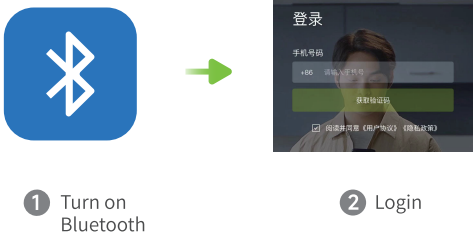

Step 2 Charge the transmitter

- Place the transmitter into the charging base for charging. The white indicator will flash (indicating charging).
- Wait for the indicator to turn blue (indicating full charging) and remove the transmitter.



2 Connect the device


Step 1 Connect to the transmitter

- On the "Connect Transmitter" page of the App, select the SN corresponding to the transmitter to complete the connection.



Step 2 Connect to the sensor

- On the App page, scan the QR code on the plastic housing of sensor and enter the initialization countdown mode.

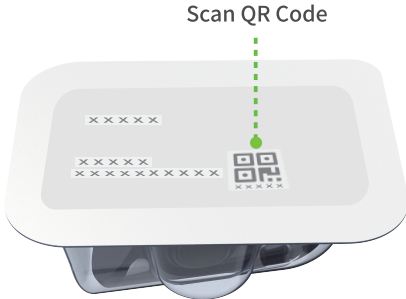


3 Implant the sensor

Step 1

Suggestion: Wear on the abdomen


- Choose an abdominal implantation (the site within 5 cm diagonally above the navel, in the dashed area shown in the figure), avoiding the belt, scars, and ribs.
- Disinfect with alcohol prep pad and air dry it before the next step.

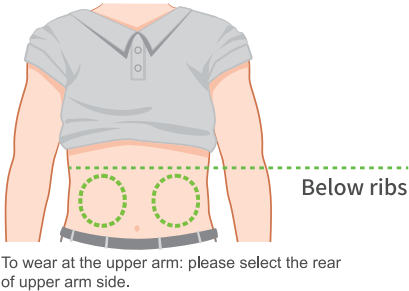

Step 2

Do not unlock before applying to the skin


- Tear off two protective films from the orange area.

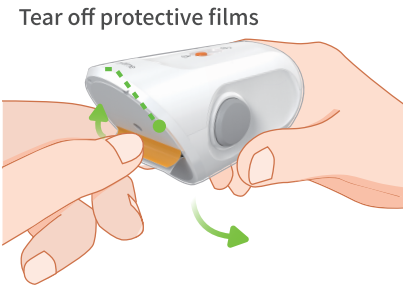

Step 3


Do not unlock before applying to the skin

- Place the sensor tightly against the implantation site.

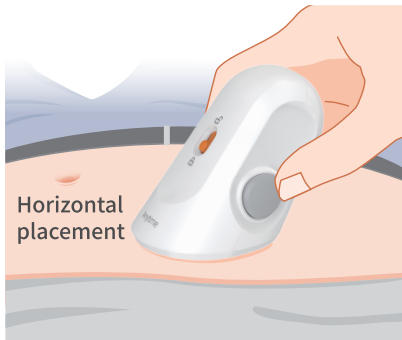


Step 4

- Slide the safety lock to unlock.

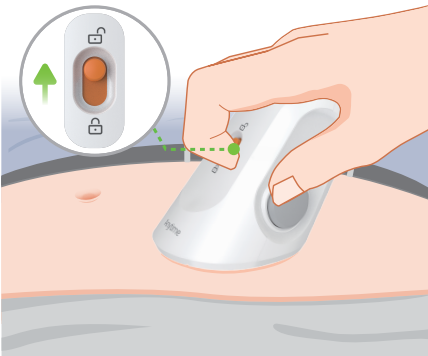


Step 5

- Press the buttons on two sides at the same time until a "click" sound is heard.

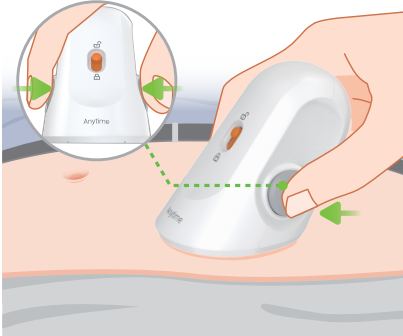

Step 6

- Slowly remove the insertion device part of the sensor to complete the implantation.

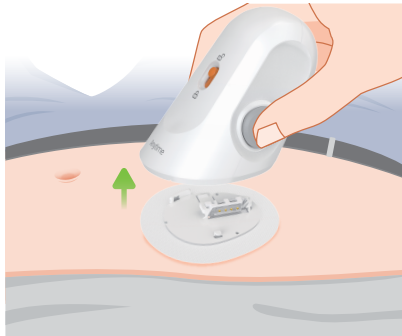

④ Install the transmitter

Step 1

★ Important steps

- Snap the transmitter onto the sensor according to the angle shown in the following figure.




Step 2


- Firmly pinch the transmitter and sensor as shown in the following figure until a "click" sound is heard.

⚠ Push the transmitter in reverse to verify it is locked.

1

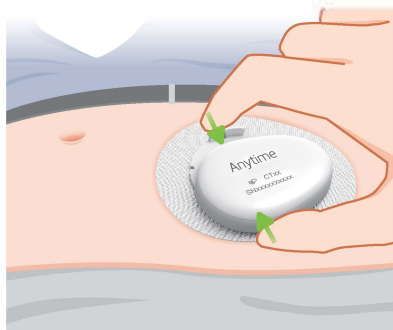


2

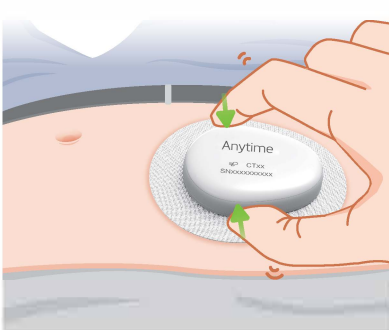


⑤ Software interface

- The App starts displaying the blood glucose value after initialization ends.

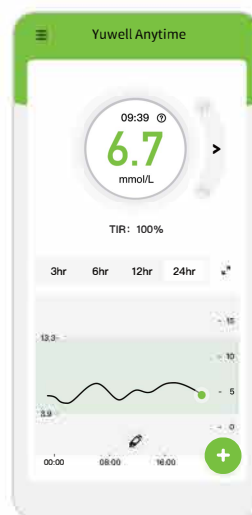

Stop monitoring and remove the device

- Slowly remove as a whole with the tape and device from the wearing site.
- After removal, force apart the buckle downwards and slide the transmitter open.
- Dispose of the sensor as medical waste.

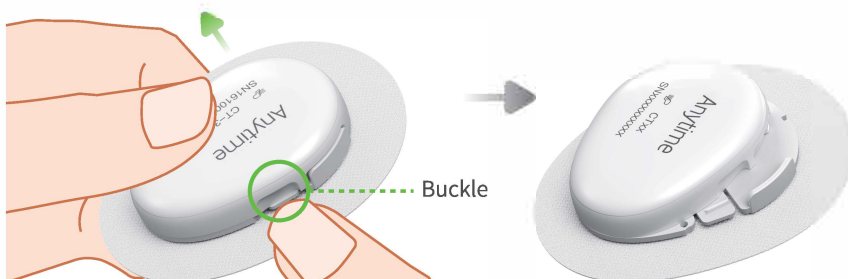

⚠ Do not throw the transmitter by mistake

⚠ The transmitter is immediately charged

Available from Essential Aids
www.essentialaids.com

customerservice@essentialaids.com

 essentialaids.com

Anytime!